


The logo for Contour Plus ONE is centered on the page. 'Contour' is in white, 'plus' is in white, and 'ONE' is in a bold, teal color with a double-stroke effect. Below 'plus ONE' is the text 'Blood Glucose Monitoring System' in a smaller white font.

# Contour<sup>®</sup> plus ONE<sup>»</sup>

Blood Glucose  
Monitoring System


Pairing your  
**CONTOUR®PLUS ONE**  
smart meter with the  
**CONTOUR™DIABETES app**

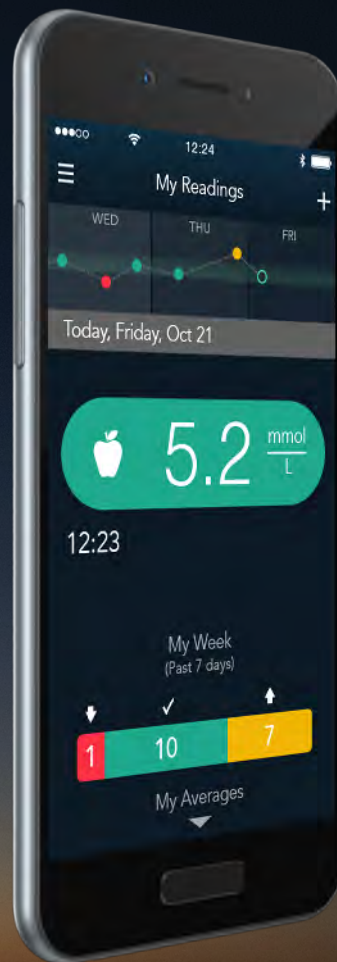

Contour  
plus **ONE**  
Blood Glucose  
Monitoring System



Pair your **CONTOUR PLUS ONE** meter and it will sync with the **CONTOUR DIABETES app** whenever a Bluetooth connection is available.

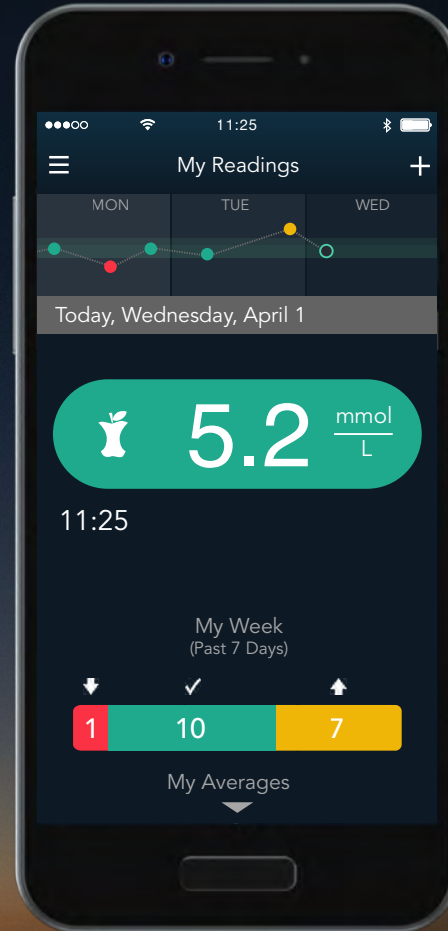


**Contour<sup>®</sup>**  
**plus ONE<sup>»</sup>**  
Blood Glucose  
Monitoring System

1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

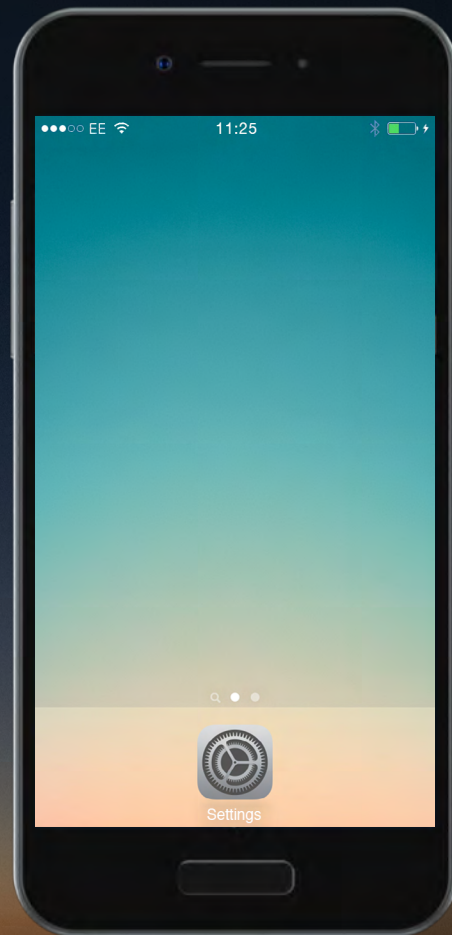

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System




1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

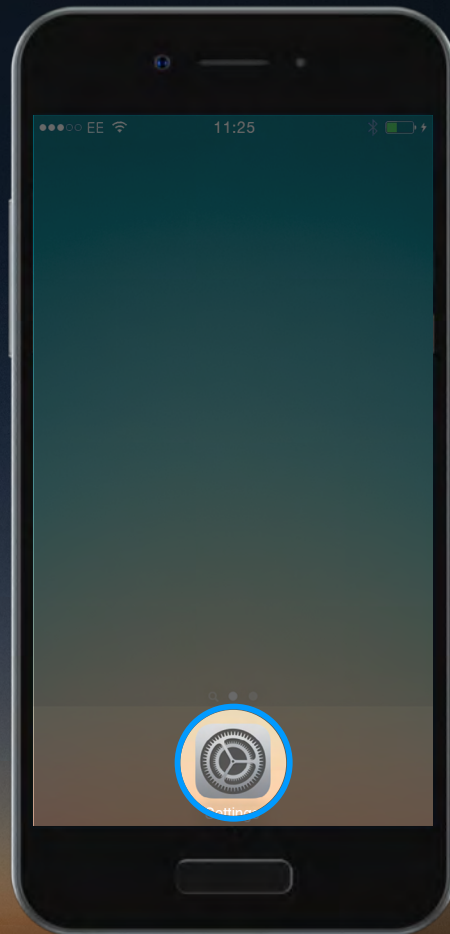


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System

1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

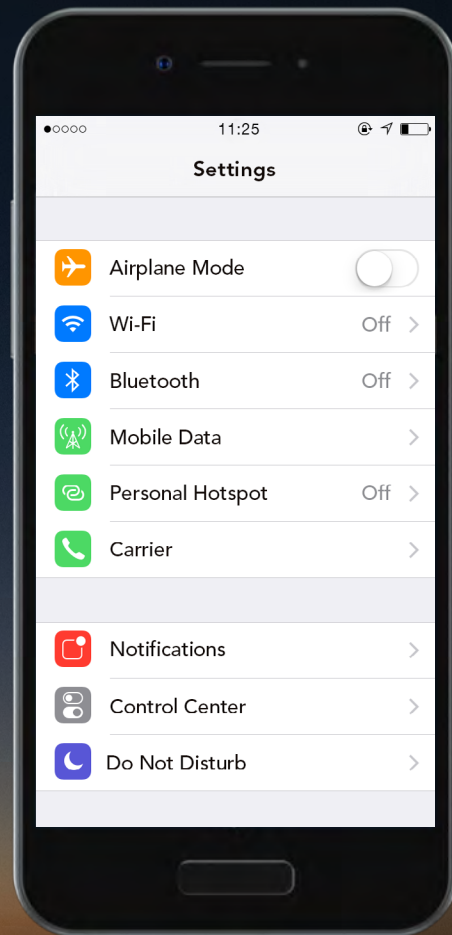

Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System




1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

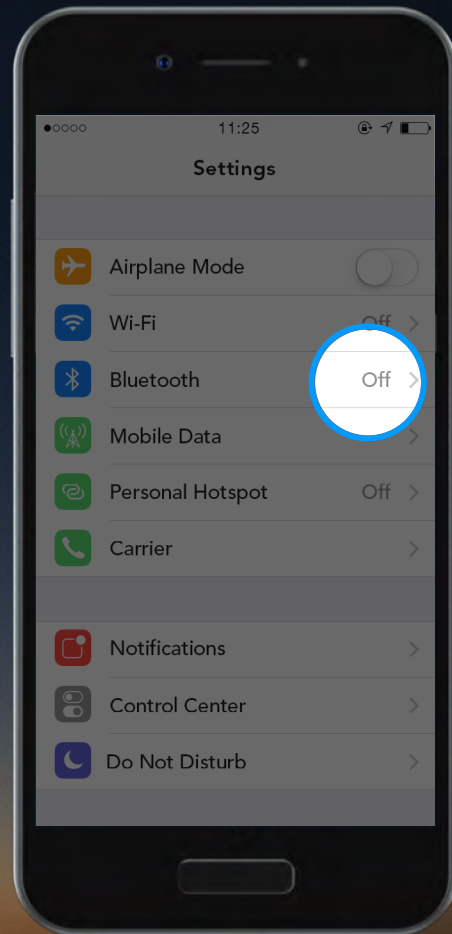


**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System

1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

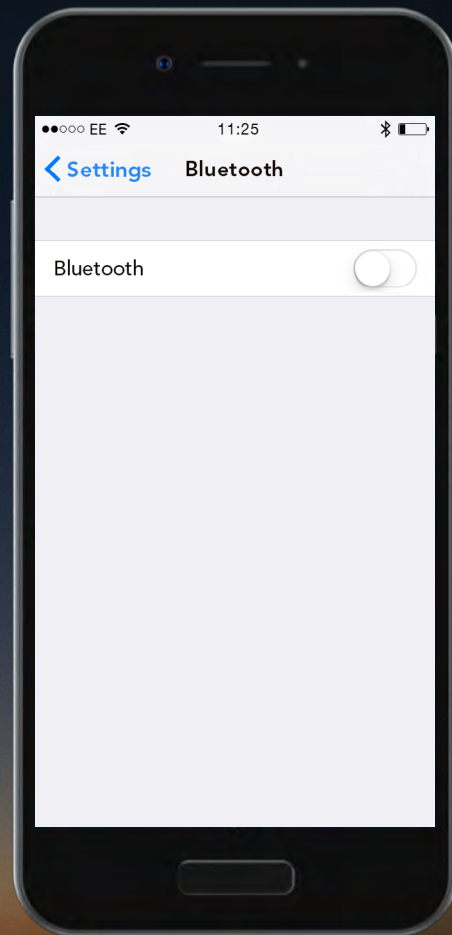

Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System




1.

## Pairing your meter

Before you pair your **CONTOUR PLUS ONE** meter, turn on Bluetooth in your mobile device.

Go to 'Settings' and then 'Bluetooth' and tap the 'toggle button' to turn on Bluetooth.

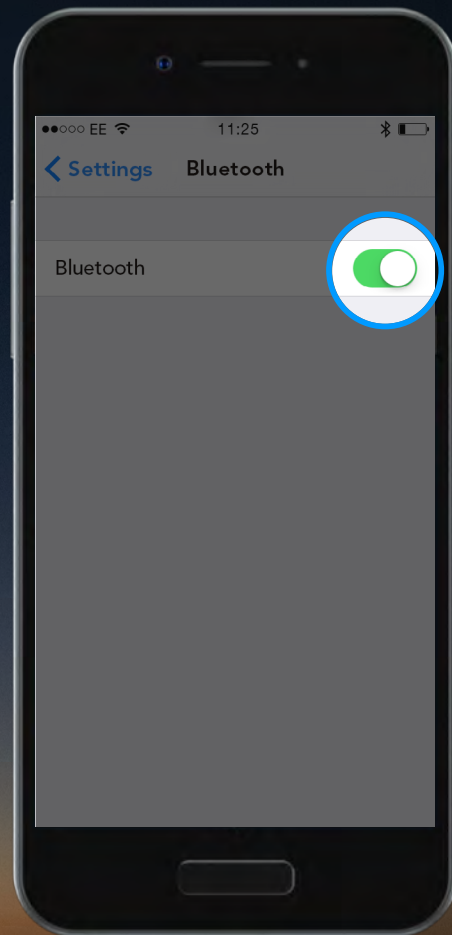

Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System


2.

## Pairing Your Meter

In the **CONTOUR DIABETES** app, after you have set up an account you will be asked to pair a meter.

You are now ready to pair your meter. Turn off your meter and keep it near your mobile device.

Select '**Continue**'.

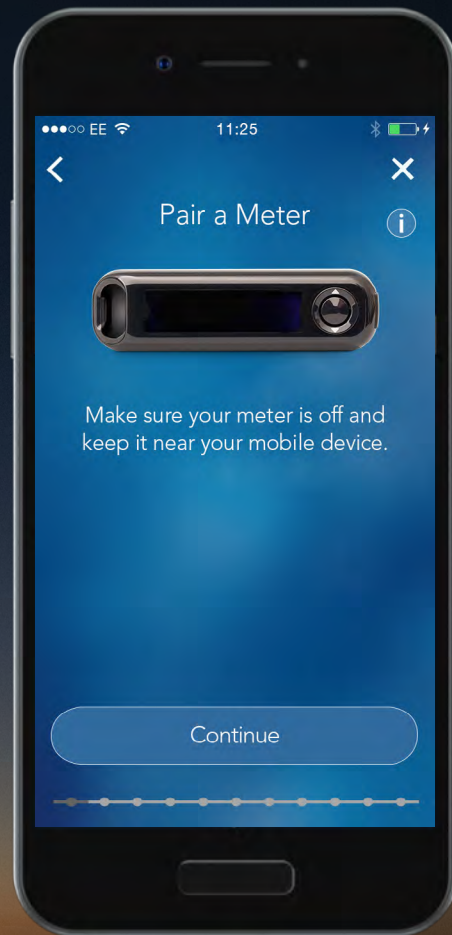

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System




2.

## Pairing Your Meter

In the **CONTOUR DIABETES** app, after you have set up an account you will be asked to pair a meter.

You are now ready to pair your meter. Turn off your meter and keep it near your mobile device.

Select 'Continue'.

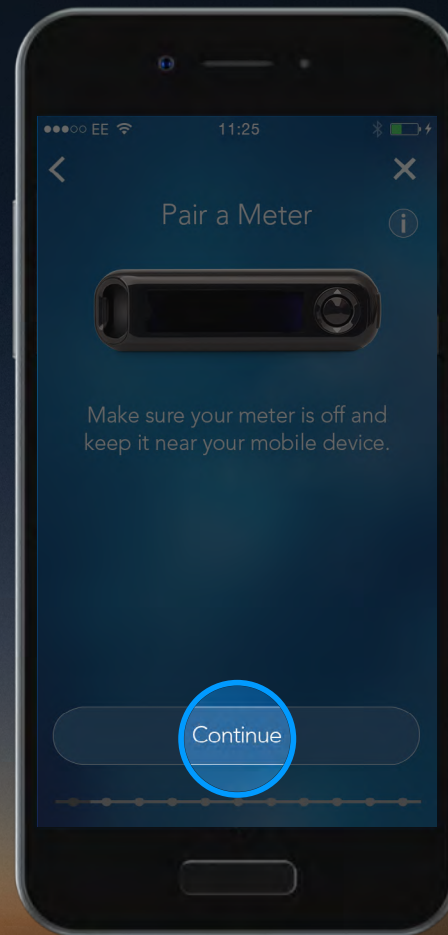


**Contour<sup>®</sup>  
plus ONE<sup>»</sup>**  
Blood Glucose  
Monitoring System

3.

### Pairing Your Meter

Press and hold the center button on your **CONTOUR PLUS ONE** meter and do not release until you see the flashing blue light from the strip port.

Select '**Continue**'.

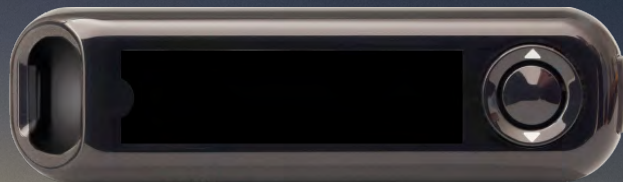

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System




3.

### Pairing Your Meter

Press and hold the center button on your **CONTOUR PLUS ONE** meter and do not release until you see the flashing blue light from the strip port.

Select '**Continue**'.

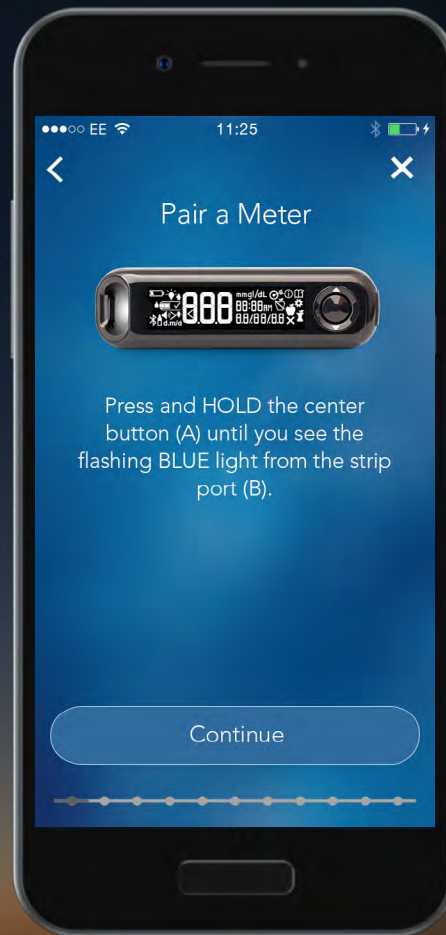


**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System

3.

### Pairing Your Meter

Press and hold the center button on your **CONTOUR PLUS ONE** meter and do not release until you see the flashing blue light from the strip port.

Select '**Continue**'.

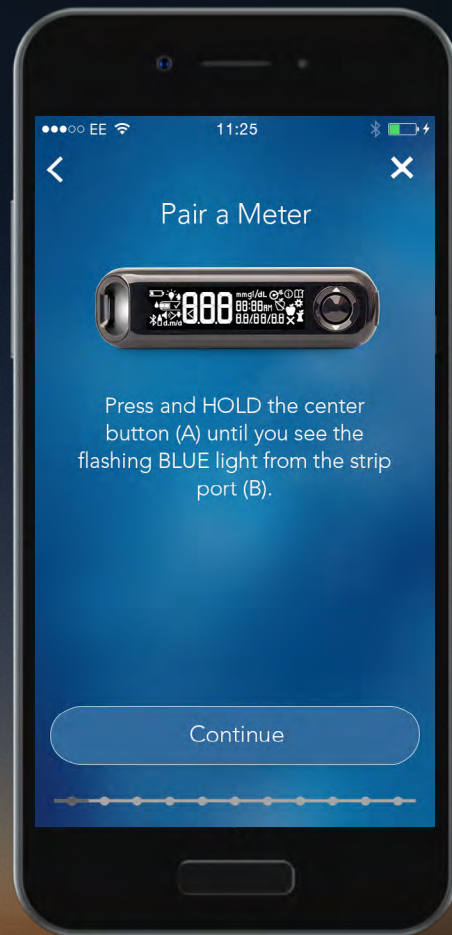

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System



3.

### Pairing Your Meter

Press and hold the center button on your **CONTOUR PLUS ONE** meter and do not release until you see the flashing blue light from the strip port.

Select '**Continue**'.



**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System


4.

## Pairing Your Meter

Select your meter's serial number.

You can find the serial number on the label on the back of your meter.

Select '**Continue**'.

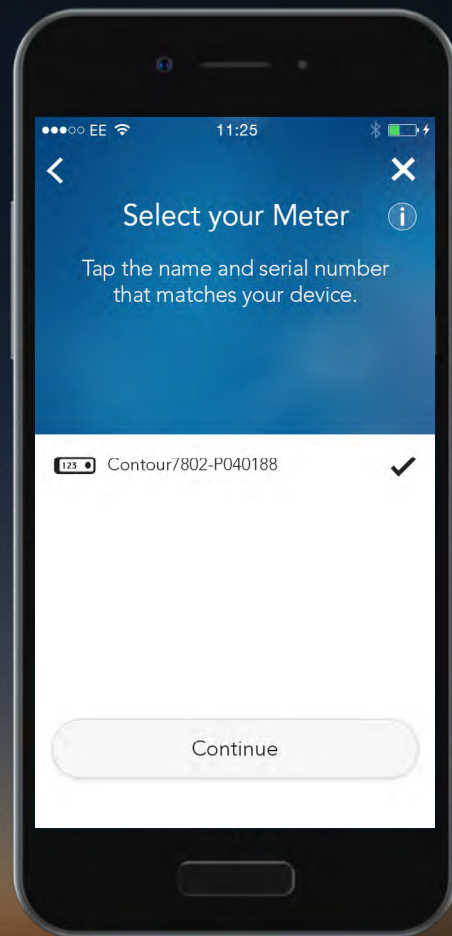

**Contour<sup>®</sup>  
plus ONE<sup>™</sup>**  
Blood Glucose  
Monitoring System




4.

## Pairing Your Meter

Select your meter's serial number.

You can find the serial number on the label on the back of your meter.

Select '**Continue**'.

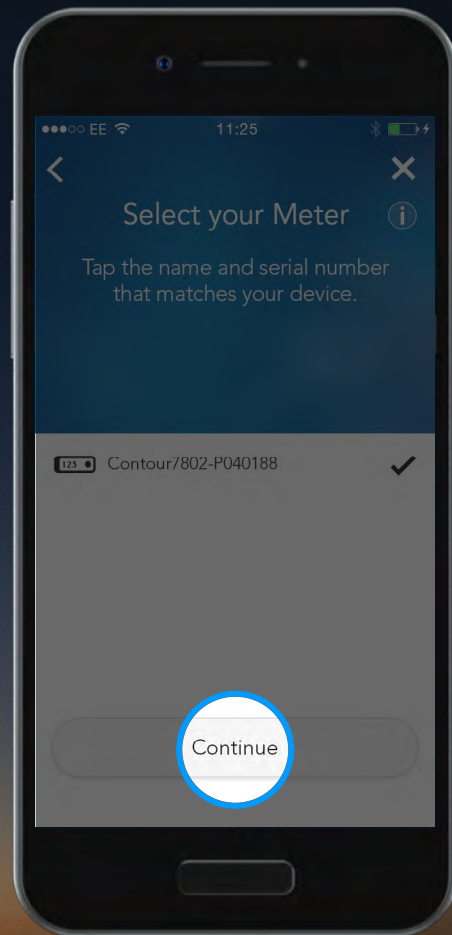


**Contour<sup>®</sup>  
plus ONE<sup>™</sup>**  
Blood Glucose  
Monitoring System

5.

## Pairing Your Meter

The **CONTOUR DIABETES** app will now connect to the meter.

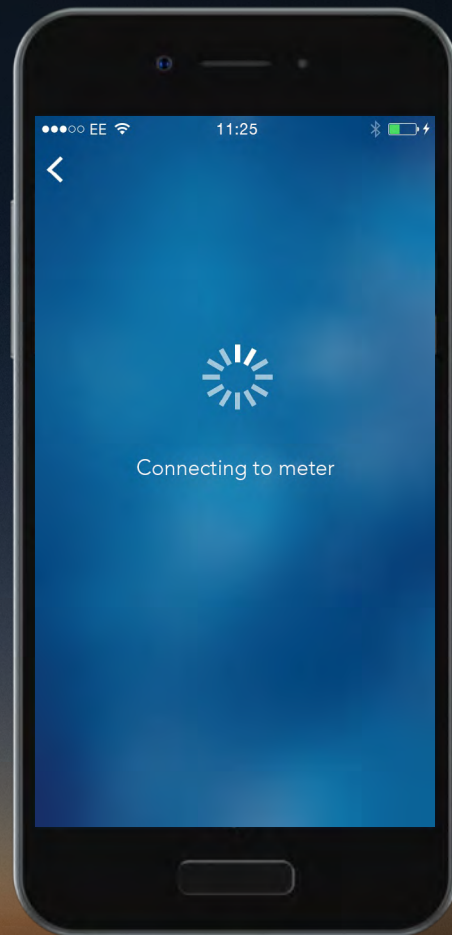


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System



6.

## Pairing Your Meter

Select 'Pair' to confirm the pairing request.

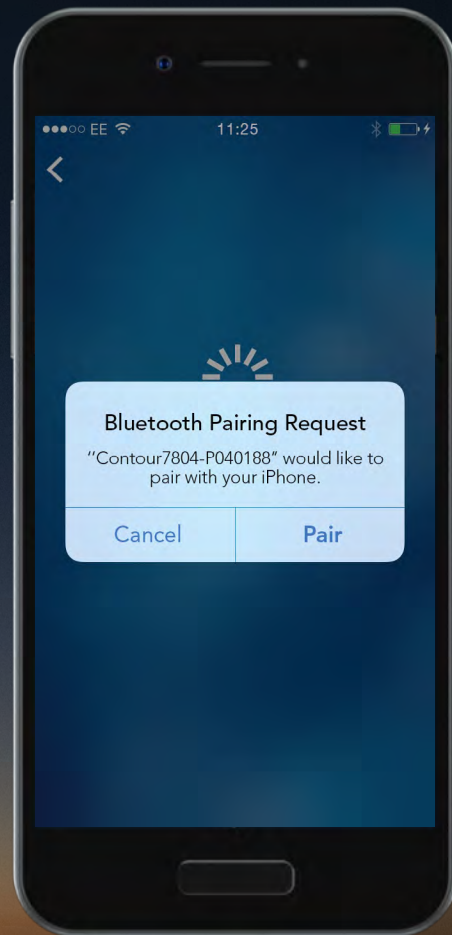


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System

6.

## Pairing Your Meter

Select 'Pair' to confirm the pairing request.

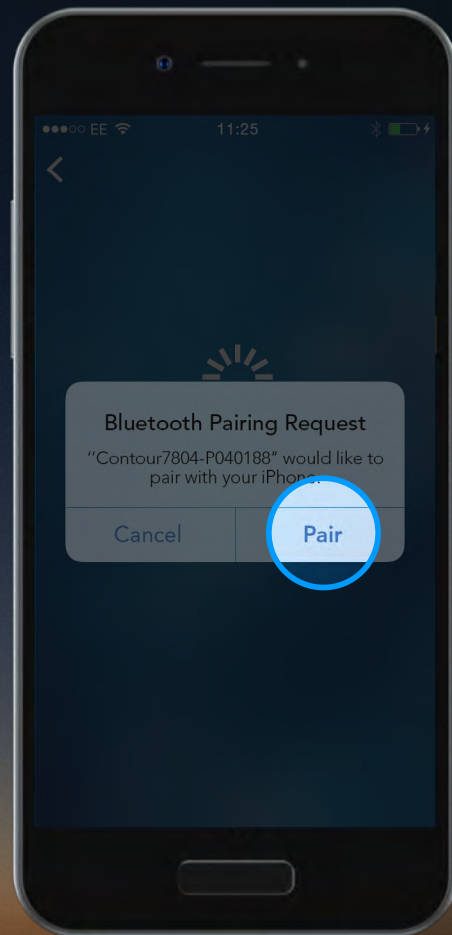


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System



6.

## Pairing Your Meter

Select 'Pair' to confirm the pairing request.

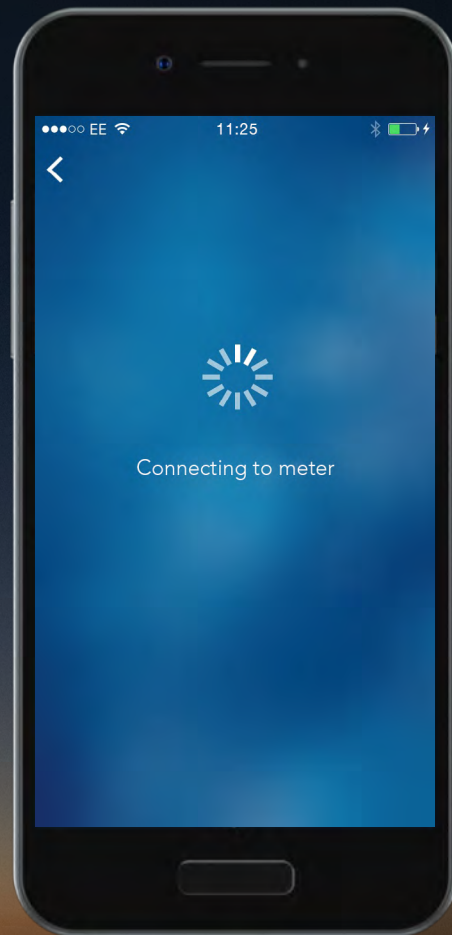


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System

7.

## Pairing Your Meter

Once your meter is paired select 'Continue'.

You are now ready to set up your  
**CONTOUR DIABETES app.**

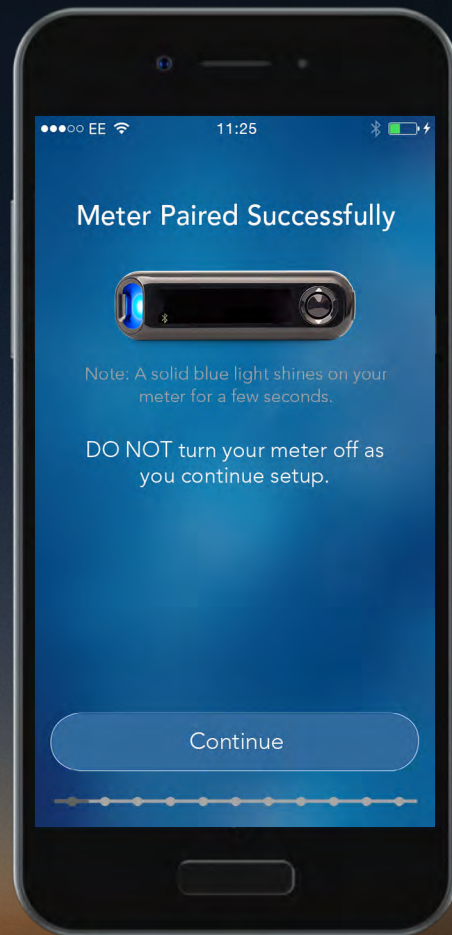

Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System




7.

## Pairing Your Meter

Once your meter is paired select 'Continue'.

You are now ready to set up your  
**CONTOUR DIABETES app.**

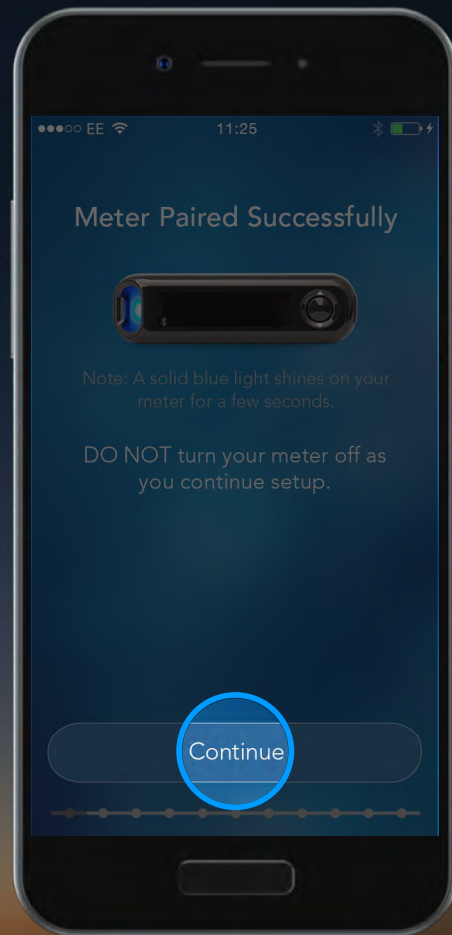


Contour<sup>®</sup>  
plus **ONE**  
Blood Glucose  
Monitoring System

Talk to your Healthcare Professional and refer to the product user guide if you have questions about the **CONTOUR®PLUS ONE** smart meter and the **CONTOUR™DIABETES app**. For more information, and to find out whether the **CONTOUR®PLUS ONE** meter is available in your country, visit [www.contourplusone.co.za](http://www.contourplusone.co.za)




Get the app

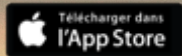

Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc.

Ascensia, the Ascensia Diabetes Care logo and CONTOUR are trademarks of Ascensia Diabetes Care Holdings AG.

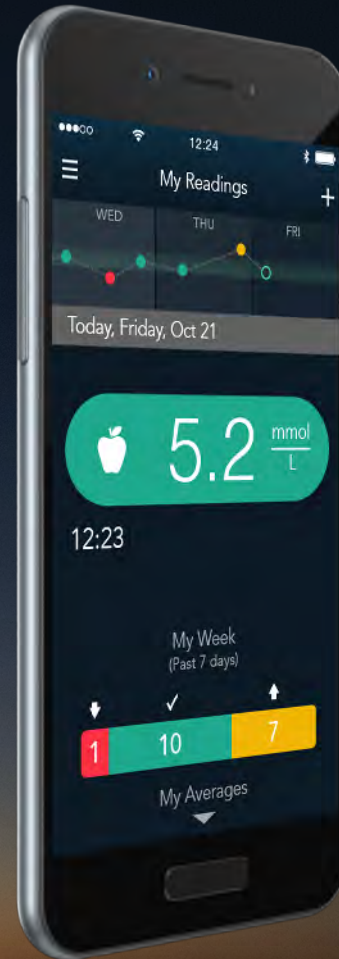

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System





# Contour<sup>®</sup>

diabetes solutions

